

Instructions for Use



Adjustable Height Armboard

- REF** Adjustable Height Armboard
800-0009
- REF** Adjustable Height Armboard (DEN)
800-0009-DEN
- REF** Adjustable Height Armboard (EU)
800-0009-EU
- REF** Adjustable Height Armboard (JPN)
800-0009-JPN
- REF** Adjustable Height Armboard (SWISS)
800-0009-SWISS
- REF** Adjustable Height Armboard (UK)
800-0009-UK

Pad sold separately: Choose one
2" Armboard Pad, Deluxe 508-0071
2" Armboard Pad, Softcare 508-0151




Extra-Wide, Adjustable Height Armboard

- REF** Extra-Wide, Adjustable Height
Armboard 800-0017
- REF** Extra-Wide, Adjustable Height
Armboard (DEN) 800-0017-DEN
- REF** Extra-Wide, Adjustable Height
Armboard (EU) 800-0017-EU
- REF** Extra-Wide, Adjustable Height
Armboard (JPN) 800-0017-JPN
- REF** Extra-Wide, Adjustable Height
Armboard (SWISS) 800-0017-SWISS
- REF** Extra-Wide, Adjustable Height
Armboard (UK) 800-0017-UK

Pad sold separately: Choose one
**2" Extra-Wide Armboard Pad, Deluxe
508-0105**
**2" Extra-Wide Armboard Pad, Softcare
508-0152**

INTENDED USE

Intended use is to hold patient's arm with range of motion of 180°. The intended users of this device are medical professionals within hospitals and surgery centers.

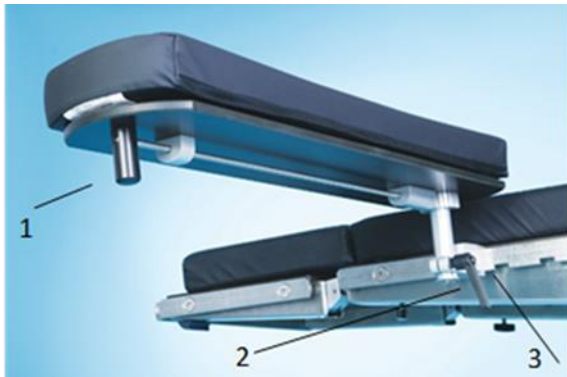

Adjustable Height Armboard



Extra-Wide Adjustable Height Armboard

(1) Directional Trigger, (2) Adjustable Handle (3) Release Trigger

INSTRUCTIONS

Become familiar with the features of patient positioning device before use with a patient. Always practice use on a nurse, physician or appropriate volunteer prior to using clinically.

Attaching Armboard to Table

1. Attaches to any point on the surgical table side rail

Adjusting Armboard

1. Loosen Adjustment Handle - raising or lowering Armboard to desired height, then lock handle
2. Pull Directional Trigger, rotate to desired position, release to lock

Detaching Armboard from Surgical Table

1. Press Release Trigger and lift Armboard from table

GENERAL SPECIFICATIONS

- Attaches to rail of surgical table at any point on rail
- Single-person installation
- Push Release Trigger

	Adjustable Height Armboard 800-0009	Extra-Wide Adjustable Height Armboard 800-0017
<i>Length</i>	26.5" +/- 0.5" (67 cm x 1 cm)	26.5" +/- 0.5" (67 cm x 1 cm)
<i>Width</i>	6" +/- 0.5" (15 cm x 1 cm)	11" +/- 0.5" (28 cm x 1 cm)
<i>Depth</i>	6.5" +/- 0.5" (17 cm x 1 cm) with pad	6.5" +/- 0.5" (17 cm x 1 cm) with pad
<i>Device Weight</i>	6 +/- 0.5 lbs. (2.7 +/- 0.22 kg)	9 +/- 0.5 lbs. (4 +/- .22 kg)

COMPONENT OVERVIEW

Adjustable Height Armboards are surgical table extensions that attach to side rails of any table.

See side rail dimensions below:

US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0009 & 800-0017

Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0009-DEN & 800-0017-DEN

Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0009-EU & 800-0017-EU

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0009-UK & 800-0017-UK

Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0009-JPN & 800-0017-JPN

Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0009-SWISS & 800-0017-SWISS

Adjustable Height Armboard 800-0009

Pad sold separately: Choose one

2" Armboard Pad, Deluxe 508-0071

2" Armboard Pad, Softcare 508-0151

Extra-Wide Adjustable Height Armboard 800-0017

Pad sold separately: Choose one

2" Extra-Wide Armboard Pad, Softcare 508-0152

2" Extra-Wide Armboard Pad, Deluxe 508-0105

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Device supports 500 lb. (227 kg) proportional patient load (6'4" (193 cm) tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



PRODUCT USE WARNINGS

WARNING!

Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. (227 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.



WARNING!

Hazards result from incorrect use. Strictly follow Instructions for Use with your Operating Table system.



WARNING! *Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.*

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.



WARNING!

Always follow the standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use the recommended protective clothing, gloves, masks and eye protection to clean the accessory.

CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

- 1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
- 2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
- 3. Let accessory dry*

USER NOTICE

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

UDI Basic UDI-DI: 081001460F0026E2

eIFU Language Versions

To download and print the Instructions for Use, please go to <http://www.schuremed.com/schuremed-eifu>.

Symbol Glossary

Symbol	Title	Symbol Description
	Manufacturer	Indicates the medical device manufacturer.
	Authorized Representative in the European Community	Indicates the authorized representative in the European community.
	Batch Code	Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Catalogue or Item Number	Indicates the manufacturer's catalogue or item number so that the medical device can be identified.
	Serial Number	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Caution	To indicate that caution is necessary when operating the device or control close to where the symbol is placed, or to indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
	Medical Device	Indicates the item is a medical device.
	Unique Device Identifier	Indicates a barcode as containing unique device identifier information.
	CE Marking	European Conformity.



Manufacturer

SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**



Authorized Representative

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands