



Instructions for Use

**Schure Foot for Total Knee Replacement,
Lexan 800-0011**

Disposable Accessory
Disposable Foot Pads 12/cs 508-0111

Copyright Notice

All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided in relevant copyright laws.



Initial Release 12/29/2004



Manufacturer

SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**



Authorized Representative

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician or appropriate volunteer prior to using clinically.

Set Up and Use of Schure Foot

- *Attach socket to surgical table side rail near end of torso section, on side of operative knee*
- *Slide black disposable foot pad #508-0111 (sold separately) onto Schure Foot tube*
- *Insert Schure Foot mounting post into socket*
- *Once patient is on surgical table, slide Schure Foot along rail to achieve desired position for knee*
- *Turn socket handle clockwise securing Schure Foot in desired position*
- *Prep and drape in usual method*
- *After completing procedure, discard the Schure Foot disposable foot pad. Spray Schure Foot with hospital-approved disinfectant and wipe clean*

INTENDED USE

Intended use is to hold patient's foot firmly in place during knee replacement surgery.

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

- *Length: 16" +/- 0.5"*
- *Diameter: 2" +/- 0.5" (with pad 2 3/4")*
- *Device Weight: 2 +/- 0.5 lbs.*
- *Operates on leg section of surgical table*
- *Requires a socket to attach to surgical table*

COMPONENT OVERVIEW

SCHURE FOOT, STAINLESS STEEL is a knee positioning device used in knee replacement surgery.

Replacement Part:

508-0111 Disposable Foot Pads 12/cs

Other required products for use: 800-0134 Schure Socket XL

US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0134

Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0134-DEN

Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0134-EU

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0134-UK

Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0134-JPN

Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0134-SWISS

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Device supports 500 lb. proportional patient load (6'4" tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F*

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



Product Use Warnings

WARNING!

Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.



WARNING!

Hazard resulting from incorrect use. Strictly follow instructions for your Operating Table system.

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.



WARNING!

Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.

CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

- 1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
- 2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
- 3. Let accessory dry*



Symbol Explanation



Date of Manufacture



Conformity Mark



Manufacturer



Batch Code/Serial Number



Authorized Representative
in European Community