



Instructions for Use

Copyright Notice

All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided in relevant copyright laws.

Lithotomy Stirrups 800-0049



Initial Release: 3/10/2007



Manufacturer

SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**



EC REP Authorized Representative

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

INSTRUCTIONS

Become familiar with the features of patient positioning device before use with a patient. Always practice use on a nurse, physician or appropriate volunteer prior to using clinically.

Attaching Stirrups to Surgical Table

1. Attach Schure Socket Clamps (sold separately) onto table side rails.
2. Insert Lithotomy Stirrup mounting posts into clamps. Turn Schure Socket handle clockwise to tighten.
3. Put patient's legs into boots and fasten with loop and hook straps.

Adjusting Stirrups

1. Turn boot block handles counterclockwise to loosen boot blocks. Adjust up or down to desired position. Turn boot block handle clockwise to tighten.

Detaching Stirrups from Table

1. Turn Schure Socket handle counterclockwise to loosen and pull stirrup from socket.

INTENDED USE

Intended use is to hold patient's legs during short or long gynecologic, urological or laparoscopic procedures.

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

- *Length 26.5" +/- 0.5"*
- *Width: 6" +/- 0.5"*
- *Depth: 2.5" +/- 0.5" (with pad)*
- *Device Weight: 5.25 +/- 0.5 lbs.*
- *Attaches to rail of surgical table at any point on the rail*
- *Single-person installation*
- *Push Release Trigger*

COMPONENT OVERVIEW

Lithotomy Stirrups are a surgical table accessory that aids in positioning of legs for gynecological, urological, or laparoscopic procedures.

Other required products for use:

800-0006 Schure Socket, 2 each (sold separately)

- *US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0006*
- *Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0006-DEN*
- *Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0006-EU*
- *Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0006-UK*
- *Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0006-JPN*
- *Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0006-SWISS*

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Device supports 350 lb. proportional patient load (6'4" tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F*

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



Product Use Warnings

WARNING!

Maximum load should not exceed appropriate proportion of a patient weighing 350 lbs. Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.



WARNING!

Hazard resulting from incorrect use. Strictly follow Instructions for Use with your Operating Table system.



WARNING!

Surgical table load capacities may be less. Never overload a surgical table. Device is intended for mounting on side of surgical table rail only.

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedure.



WARNING!

Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.

CAUTION

Strictly read/follow manufacturer’s directions for cleaning fluids. DO NOT use cleaners containing phenolics.

1. *Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
2. *Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
3. *Let accessory dry*



Symbol Explanation



Date of Manufacture



Conformity Mark



Manufacturer



Batch Code/Serial Number



Authorized Representative
in European Community