



Instructions for Use

REF Schure Spine Frame Dolly 800-0074-SF

INTENDED USE

Intended use is to store components of our Schure Spine Frame. The intended users of this device are medical professionals within hospitals and surgery centers.

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

- *Length: 27" +/- 0.5" (69 cm +/- 1 cm)*
- *Width: 12" +/- 0.5" (31 cm +/- 1 cm)*
- *Height: 39" +/- 0.5" (99 cm +/- 1 cm)*
- *Device Weight: 28 +/- 0.5 lbs. (12.7 +/- .22 kg)*
- *Fully Assembled*

COMPONENT OVERVIEW


Schure Spine Frame Dolly is a storage cart used to store components of our Schure Spine Frame.

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



INSTRUCTIONS

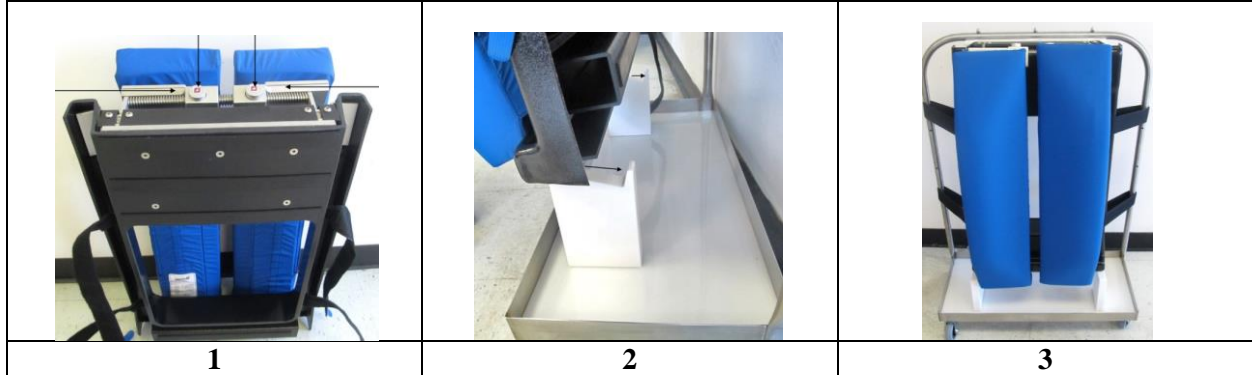

Step 1. When placing Spine Frame onto Dolly, have buttons facing up and close together as shown.

Step 2. Place Spine Frame onto Dolly (as shown) and slide all the way back until plastic of frame aligns with back of Dolly blocks.

Step 3. Loop hook and loop fastener straps around Dolly and fasten them onto frame.



PRODUCT USE WARNINGS

WARNING!

Hazard resulting from incorrect use. Be absolutely sure to follow Instructions for Use for when operating Dolly.



WARNING! Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.

CLEANING & DISINFECTING

Spray and wipe clean with hospital approved disinfectant. No sterilization.

USER NOTICE






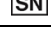





Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

UDI Basic UDI-DI: 081001460F0006DU

eIFU Language Versions

To download and print the Instructions for Use, please go to <http://www.schuremed.com/schuremed-eifu>.

Symbol Glossary

Symbol	Title	Symbol Description
	Manufacturer	Indicates the medical device manufacturer.
	Authorized Representative in the European Community	Indicates the authorized representative in the european community.
	Use-by Date	Indicates the date after which the medical device is not to be used.
	Batch Code	Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Catalogue or Item Number	Indicates the manufacturer's catalogue or item number so that the medical device can be identified.
	Serial Number	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Caution	To indicate that caution is necessary when operating the device or control close to where the symbol is placed, or to indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
	Medical Device	Indicates the item is a medical device.
	Unique Device Identifier	Indicates a barcode as containing unique device identifier information.
	CE Marking	European Conformity.
	Single Patient Use	Indicates the item is a single patient use medical device.



Manufacturer

SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**



Authorized Representative

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands