



## Instructions for Use

### **REF** Shoulder Chair Dolly 800-0074

#### **INTENDED USE**

Intended use is for storage of SchureMed's shoulder chair. The intended users of this device are medical professionals within hospitals and surgery centers.

#### **INSTRUCTIONS**

1. Place Shoulder Chair onto mounting blocks as shown
2. Secure head section with hook & loop fastener strap

#### **GENERAL SPECIFICATIONS**

*Device Dimensions (maximum)*

- Length: 27" +/- 0.5" (69 cm +/- 1 cm)
- Width: 12" +/- 0.5" (31 cm +/- 1 cm)
- Height: 39" +/- 0.5" (99 cm +/- 1 cm)
- Device Weight: 28 +/- 0.5 lbs. (12.7 +/- .22 kg)
- Fully Assembled



#### **COMPONENT OVERVIEW**

Shoulder Chair Dolly is a storage cart used to store shoulder chair components.

#### **GENERAL INFORMATION**

- Product not made with Natural Rubber Latex
- Product warranty covers product from manufacturing defects for period of 2 years
- If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.
- CE marked medical device according to MDR (EU) 2017/745
- Product is maintenance-free, check product condition before next use
- Life of device is 5 years under normal use
- Store device between -4°F to +86°F (-20°C to 30°C)

#### **DISPOSAL**

- **General** - Prevent infection by cleaning and disinfecting product before disposal
- **Packaging** - Dispose packaging material via household waste according to national requirements
- SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements



### PRODUCT USE WARNINGS

**WARNING!** Hazard resulting from incorrect use. Be absolutely sure to follow Instructions for Use when operating dolly.



**WARNING!** Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.

### CLEANING & DISINFECTING

Spray and wipe clean with hospital-approved disinfectant. No sterilization.

### USER NOTICE

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**UDI** Basic UDI-DI: 081001460F0006DU

### eIFU Language Versions

To download and print the Instructions for Use, please go to <http://www.schuremed.com/schuremed-eifu>.

### Symbol Glossary

| Symbol | Title   | Symbol Description  |
|--------|---|---|
|        | Manufacturer  | Indicates the medical device manufacturer.  |
|        | Authorized Representative in the European Community | Indicates the authorized representative in the European community.  |
|        | Use-by Date   | Indicates the date after which the medical device is not to be used.  |
|        | Batch Code  | Indicates the manufacturer's batch code so that the batch or lot can be identified.   |
|        | Catalogue or Item Number                            | Indicates the manufacturer's catalogue or item number so that the medical device can be identified.   |
|        | Serial Number                                       | Indicates the manufacturer's serial number so that a specific medical device can be identified.   |
|        | Caution   | To indicate that caution is necessary when operating the device or control close to where the symbol is placed, or to indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences. |
|        | Medical Device                                      | Indicates the item is a medical device.   |
|        | Unique Device Identifier                            | Indicates a barcode as containing unique device identifier information.   |
|        | CE Marking  | European Conformity.  |
|        | Single Patient Use                                  | Indicates the item is a single patient use medical device.  |




### Manufacturer

SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | Ph (781) 982-7000 | Fax (781) 982-7001 | [orders@schuremed.com](mailto:orders@schuremed.com)

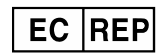

**Authorized Representative**

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands