



Instructions for Use

TKR Positioner, 24" (61 cm) 800-0141

TKR Positioner, 30" (76 cm) 800-0171

Replacement Liners

TKR Boot Liners, 12/cs 800-0146

Copyright Notice

All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided for in relevant copyright laws.

Device Initial Release: 11/17/2009



Manufacturer

SchureMed (081001460)
452 Randolph St., Abington, MA 02351 USA
Toll Free (888) 724-8763
Ph (781) 982-7000 | Fax (781) 982-7001
orders@schuremed.com

EC REP Authorized Representative

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands


GENERAL INFORMATION

- Product not made with Natural Rubber Latex
- Device supports 500 lbs. (227 kg) (proportional patient load 6'4" (193 cm) (tall patient per 99% human body model)
- Product warranty covers product from manufacturing defects for period of 2 years
- If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.
- Product is maintenance-free, check product condition before next use
- CE marked medical device according to MDR (EU) 2017/745
- Life of device is 5 years under normal use
- Store device between -4°F to $+86^{\circ}\text{F}$ (-20°C to $+30^{\circ}\text{C}$)
- All components must be cleaned before and after use. See cleaning instructions on page 4 of this manual.
- All components need to be steam sterilized. See sterilization instructions on page 4-6 of this manual.



INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician, or appropriate volunteer prior to using clinically.

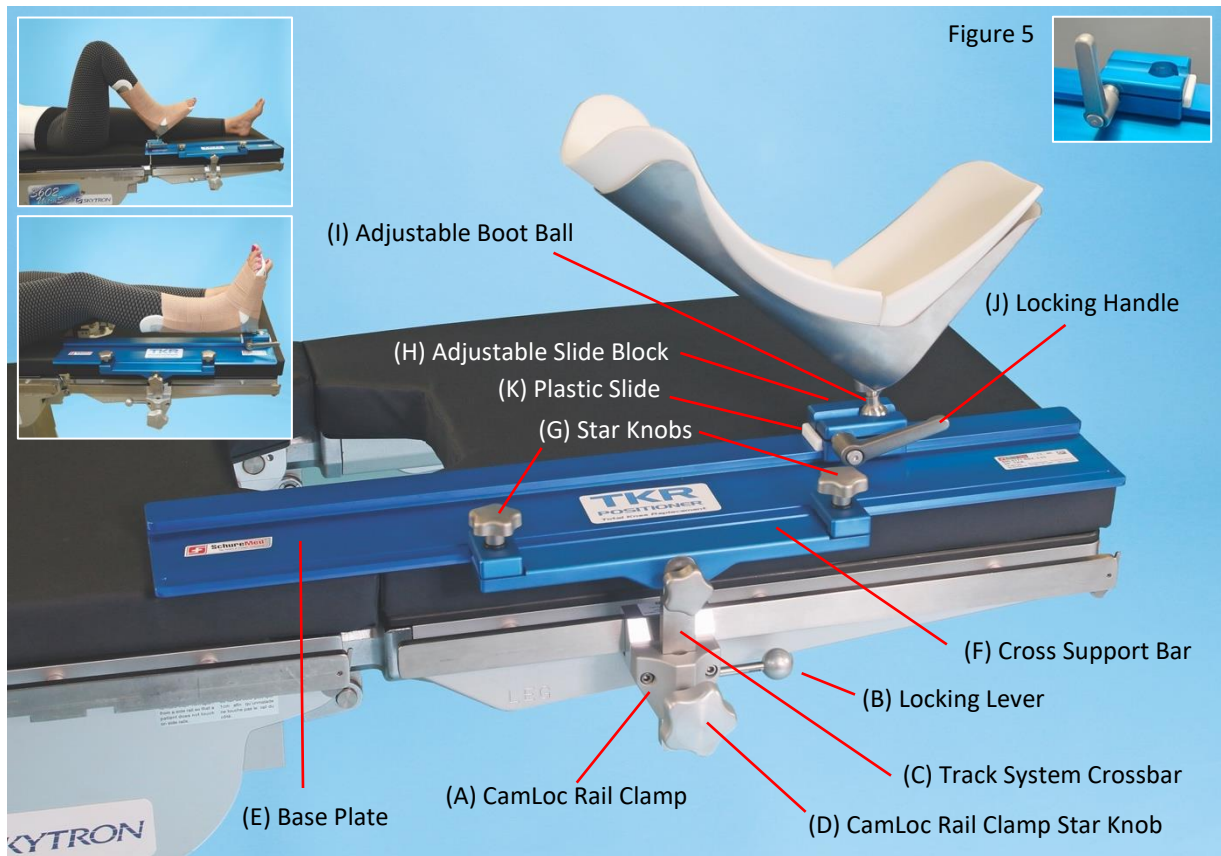

Set Up and Use of TKR Positioner

1. *Drape surgical table. Choose Standard CamLoc Rail Clamp (sold separately) to use with 1-2 drapes or Large CamLoc Rail Clamp for 3-5 drapes.*
2. *Attach CamLoc Rail Clamp (A) to rail. Then flip Locking Lever (B) right or left to secure. **(DO NOT ATTACH OVER NOTCHES WITHIN SIDE RAIL).***
3. *Slide Track System Crossbar (C) into CamLoc Rail Clamp (A). Securely tighten Track System Crossbar (C) by turning CamLoc Rail Clamp Star Knob (D) clockwise.*
4. *Slide Base Plate (E) into Cross Support Bar (F) making sure its flush. Tighten two Star Knobs (G) by turning clockwise.*
5. *Insert Plastic Slide (K) within opening under Adjustable Slide Block (H), with notched side up. Make sure Locking handle (J) is rotated until it stops (do not tighten) before sliding on to Base Plate (E). Locking Handle (J) should be in 12:00 position (see Figure 5). Depress button and pull Locking Handle (J) to adjust as needed. Slide on to track system while holding Plastic Slide (K) up close to Adjustable Slide Block (H). **WARNING! Only adjust adjustable slide block Locking Handle (J) lever when it is on track.***
6. *Place Adjustable Boot Ball (I) into track system's Adjustable Slide Block (H).*
7. *Line boot with our TKR Boot Liner 800-0146 (12/cs, sold separately)*
8. *Prep and position patient, turn Adjustable Slide Block Locking Handle (J) right to lock into desired position*

INTENDED USE

Intended use is to allow surgeon to position operative leg in multiple positions (extension, flexion, tilt, and rotation) during total knee replacement surgery.

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

800-0141 TKR, 24" (61 cm)

Length: 23.7" +/- 0.5" (61 cm +/- 1 cm)

Width: 7.5" +/- 0.5" (19 cm +/- 1 cm)

Height: 8.1" +/- 0.5" (20 cm +/- 1 cm)

Net Weight: 9 +/- 0.5 lbs. (4 kg +/- .22 kg)

Gross Weight: 12.5 +/- 0.5 lbs. (5.6 kg +/- .22 kg)

800-0171 TKR, 30" (76 cm)

Length: 29.7" +/- 0.5" (76 cm +/- 1 cm)

Width: 7.5" +/- 0.5" (19 cm +/- 1 cm)

Height: 8.1" +/- 0.5" (20 cm +/- 1 cm)

Device Weight: 10 +/- 0.5 lbs. (4.5 kg +/- .22 kg)

Gross Weight: 14.5 +/- 0.5 lbs. (6.5 kg +/- .22 kg)

Single-person installation.

COMPONENT OVERVIEW

TKR Positioner, 24" (61 cm) or 30" (76 cm), is a surgical accessory that allows surgeon to position operative leg during total knee replacement surgery.

800-0141-TRAY TKR Sterilization Tray, 24" (61 cm) (included with 800-0141 purchase)

800-0171-TRAY TKR Sterilization Tray, 30" (76 cm) (included with 800-0171 purchase)

Required Accessories: CamLoc Rail Clamp and TKR Boot Liners sold separately.

CamLoc Rail Clamp: 800-0271 For use with 1-2 drapes | **800-0271-L** For use with 3-5 drapes

US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0271 or 800-0271-L

Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0271-DEN

Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0271-EU

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0271-UK

Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0271-JPN

Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0271-SWISS

Replacement Liners: 800-0146 TKR Boot Liners, 12/cs

DISPOSAL

- **General** - Prevent infection by cleaning and disinfecting product before disposal
- **Packaging** - Dispose packaging material via household waste according to national requirements
- SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements



PRODUCT USE WARNINGS

WARNING!

Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. (227 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.



WARNING!

Hazards result from incorrect use. Strictly follow Instructions for Use with your Operating Table system.



WARNING! Only adjust adjustable slide block lever when it is on the track.

HAND AND WASHER MACHINE CLEANING INSTRUCTIONS

- Clean TKR Positioner thoroughly before using (before sterilization) and after each use
- Recommended cleaning products: (All solutions should be used in accordance with manufacturer’s instructions)
 - Enzymatic neutral pH detergent solution
 - (Enzymatic cleaners are compatible with metal devices/instruments and are preferred)
 - Non-enzyme, hydrogen peroxide based solution
 - Coverage Plus NPD One Step Cleaner, Disinfectant & Deodorizer Concentrate
- Disassemble TKR Positioner into separate parts for cleaning:
 - Boot Ball
 - Mounting Plate with Clamp
 - Base Plate with Track
 - Slide Block with Handle
 - Plastic Slide

HAND CLEANING

- Scrub each piece with a soft, plastic scrub brush and a recommended cleaning solution (see above)
- Rinse all parts thoroughly to remove any chemical residue
- Wipe each part completely dry with a clean, lint-free cloth

WASHER CLEANING

- Put through washer/decontaminator according to manufacturer’s instruction (Select cycle that does not include lubrication)
- Prepare for sterilization (device must be cleaned before sterilization)
- See sterilization instructions below


**Based on information provided by the Healthcare Infection Control Practices Advisory Committee (HICPAC), the Center for Disease Control (CDC), the National Center for Biotechnology Information (NCBI) and the U.S. National Library of Medicine.*

STEAM STERILIZATION

Sterilization Tray w/Lid	
<p>Outside View</p> <p>24” (61 cm) Tray Dimensions: 26.5”L x 10.5”W x 6.75”D (67 cm x 27 cm x 17 cm) Weight: 7.9 lbs. (3.6 kg)</p> <p>30” (75 cm) Tray Dimensions: 31.25”L x 10.5”W x 6.75”D (79 cm x 27 cm x 17 cm) Weight: 8.7 lbs. (3.9 kg)</p>	

Sterilization Tray

Inside View

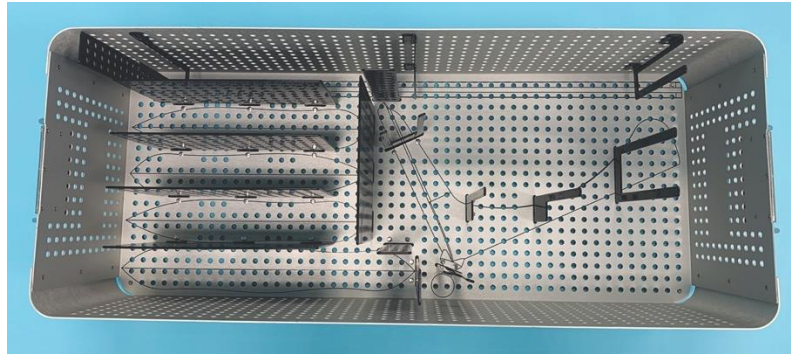

Component Placement in Pouches

7" x 12" (18 cm x 31 cm)
Pouches

Split parts evenly between 3
pouches.

- *Blade/Star Knobs*
- *Handle/Ball Slide Block
and Plastic Slide*
- *Clamps and Handle*



8" x 16" (20 cm x 41 cm)
Pouch

Mount Plate



Sterilization Tray Component Placement

Place following parts in the
dedicated slots:

1. *Place Base Plate upright
(as shown)*
2. *Place TKR Boot within
black outline for exact
placement*
3. *Place pouches vertically in
slots*



Steam Sterilization Procedure

Wrap using envelope method or accepted wrap method at your facility

Pre-Vacuum Wrapped Instruments

*Exposure Time at 270°F:
(132°C) 4 minutes*

Drying Time: 40 minutes



USER NOTICE

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.



IFU-800-0141_800-0171 REV 3.07

Basic UDI-DI: 081001460F0015DV

Latest Revision: 9/15/2021

To View Available Translations

Go to schuremed.com, select product, scroll down and click on “Instructions for Use”.