

## Instructions for Use



### Set Up and Use of the TKR Positioner

1. Drape the surgical table.
2. Attach the rail clamp to the rail (a). Then flip the locking lever (b) right or left to secure. **(DO NOT ATTACH OVER THE NOTCHES WITHIN THE SIDE RAIL).**
3. Slide the track system crossbar (c) into the rail clamp, approximately 2" from patient's heel. Securely tighten the track system crossbar by turning the rail clamp star knob to the right (d).
4. Slide the base plate (e) into the cross support bar (f) making sure its flush. Tighten the two star knobs (g) by turning to the right.
5. Slide the adjustable slide block (h) on to the track system. **WARNING! Only adjust the adjustable slide block lever when it is on the track.**
6. Place the adjustable boot ball (i) into the track system's adjustable slide block.
7. Line the boot with our sterile disposable boot liner #800-0146 (sold separately as a case of 12).
8. Prep and position patient, turn the slide block locking handle (j) right to lock into desired position.