



Instructions for Use

| | | |
|------------|----------------------------------|-----------------------|
| REF | CamLoc Rail Clamp | 800-0271 |
| REF | CamLoc Rail Clamp, Large | 800-0271-L |
| REF | CamLoc Rail Clamp (DEN) | 800-0271-DEN |
| REF | CamLoc Rail Clamp (EU) | 800-0271-EU |
| REF | CamLoc Rail Clamp (JPN) | 800-0271-JPN |
| REF | CamLoc Rail Clamp (UK) | 800-0271-UK |
| REF | CamLoc Rail Clamp (SWISS) | 800-0271-SWISS |



INTENDED USE

Intended use for CamLoc Rail Clamp is to hold accessories to surgical table. The intended users of this device are medical professionals within hospitals and surgery centers.

GENERAL INFORMATION





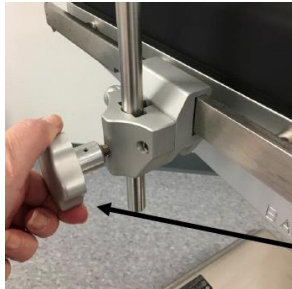
- *Choose Standard clamp (800-0271) to use with 1-2 drapes or Large clamp (800-0271-L) to use 3-5 drapes*
- *Product not made with Natural Rubber Latex*
- *Device supports 500 lb. (227 kg) proportional patient load (6'4" (193 cm) tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

- *Length: 4.75" +/- 0.5" (12 cm +/- 1 cm)*
- *Width: 5" +/- 0.5" (13 cm +/- 1 cm)*
- *Depth: 2.25" +/- 0.5" (6 cm +/- 1 cm)*
- *Device Weight: 2 +/- 0.5 lbs. (.9 kg +/- .22 kg)*
- *Attaches to rail of surgical table at any point on rail*
- *Single-person installation*

INSTRUCTIONS

| | | |
|---|---|---|
|  | <p>Step 1. <i>Place clamp onto rail (as shown).</i></p> <p>CamLoc locks to rail from the bottom. Clamp cannot be attached where there is a notch on side rail—it will not be secured or locked.</p> |  |
|  | <p>Step 2. <i>Lock clamp by turning stainless steel ball clockwise or counterclockwise (as shown).</i></p> | |
|  | <p>Step 3. <i>Inset accessory (as shown).</i></p> | |
|  | <p>Step 4. <i>Lock accessory by turning star knob to clockwise (as shown).</i></p> | |

COMPONENT OVERVIEW

CamLoc Rail Clamp is a surgical table clamp that attaches to side rails of any table.

See side rail dimensions below:

US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0271

Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0271-DEN

Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0271-EU

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0271-UK

Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0271-JPN

Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0271-SWISS

DISPOSAL

- **General** - Prevent infection by cleaning and disinfecting product before disposal
- **Packaging** - Dispose packaging material via household waste according to national requirements
- SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements



PRODUCT USE WARNINGS

WARNING!

Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. (227 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded. Strictly follow Instructions for Use for your surgical table system.



WARNING! Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.

CLEANING INSTRUCTIONS

Clean CamLoc Rail Clamp thoroughly before using & sterilization and after each use

- *Recommended cleaning products: (All solutions should be used in accordance with manufacturer's instructions)*
 - Enzymatic neutral pH detergent solution
 - (Enzymatic cleaners are compatible with metal devices/instruments and are preferred)
 - Non-enzyme, hydrogen peroxide based solution
 - Coverage Plus NPD One Step Cleaner, Disinfectant & Deodorizer Concentrate

HAND CLEANING

- *Scrub each piece with a soft, plastic scrub brush and a recommended cleaning solution (see above)*
- *Rinse all parts thoroughly to remove any chemical residue*
- *Wipe each part completely dry with a clean, lint-free cloth*

WASHER CLEANING

- *Put through washer/decontaminator according to manufacturer's instruction (Select cycle that does not include lubrication)*
- *Prepare for sterilization (device must be cleaned before sterilization)*
- *See sterilization instructions below*

**Based on information provided by the Healthcare Infection Control Practices Advisory Committee (HICPAC), the Center for Disease Control (CDC), the National Center for Biotechnology Information (NCBI) and the U.S. National Library of Medicine.*

STEAM STERILIZATION

Wrap using envelope method or accepted wrap method at your facility.

Pre-Vacuum

Wrapped Instruments

Exposure Time at 132°C (270°F): 4 minutes

Drying Time: 40 minutes

USER NOTICE






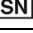





Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

UDI Basic UDI-DI: 081005737F0018KY

eIFU Language Versions

To download and print the Instructions for Use, please go to <http://www.schuremed.com/schuremed-eifu>.

Symbol Glossary

| Symbol | Title | Symbol Description |
|---|---|---|
|  | Manufacturer | Indicates the medical device manufacturer. |
|  | Authorized Representative in the European Community | Indicates the authorized representative in the European community. |
|  | Use-by Date | Indicates the date after which the medical device is not to be used. |
|  | Batch Code | Indicates the manufacturer's batch code so that the batch or lot can be identified. |
|  | Catalogue or Item Number | Indicates the manufacturer's catalogue or item number so that the medical device can be identified. |
|  | Serial Number | Indicates the manufacturer's serial number so that a specific medical device can be identified. |
|  | Caution | To indicate that caution is necessary when operating the device or control close to where the symbol is placed, or to indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences. |
|  | Medical Device | Indicates the item is a medical device. |
|  | Unique Device Identifier | Indicates a barcode as containing unique device identifier information. |
|  | CE Marking | European Conformity. |
|  | Single Patient Use | Indicates the item is a single patient use medical device. |



Manufacturer

SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | Ph (781) 982-7000 | Fax (781) 982-7001 | orders@schuremed.com



Authorized Representative



Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands