



## Instructions for Use

### Stirrup Clamp

800-0396

800-0396-EU

### Copyright Notice

*All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided for in relevant copyright laws.*

**Initial Release:** 6/6/2023



**SchureMed** (081001460)

452 Randolph Street, Abington, MA 02351 USA

**Toll Free** (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**

### GENERAL SPECIFICATIONS

*Device Dimensions (maximum)*

- *Length:* 2.75" +/- 0.5" (7 cm +/- 1 cm)
- *Width:* 1.94" +/- 0.5" (4.9 cm +/- 1 cm)
- *Depth:* 2.98" +/- 0.5" (6.8 cm +/- 1 cm)
- *Handle Collapsed:* 2.75" +/- 0.5" (7 cm +/- 1 cm)
- *Handle Extended:* 3.0" +/- 0.5" (7.62 cm +/- 1 cm)
- *Device Weight:* 0.5 +/- 0.5 lbs. (.22 kg +/- .22 kg)
- *Attaches to surgical table anywhere on rail*
- *Single-person installation*

### INTENDED USE

Intended use is to hold operating room accessories on surgical table. Clamp is primarily used together with our Great White Stirrups.

### INSTRUCTIONS

Become familiar with the features of patient positioning device before use with a patient. Always practice use on a nurse, physician or appropriate volunteer prior to using clinically.

Stirrup Clamp is spring-loaded and attaches anywhere along surgical table's side rail. Accepts 1" x 1/4" (2.5 cm x .64 cm) flat bar mounting post. Holds all accessories in vertical position.





### **Attaching to Side Rail**

1. Place clamp on rail so flat bottom is in down position
2. Push clamp down on rail until it attaches to the rail.
3. Secure clamp by turning handle clockwise to locked position

### **Detaching from Side Rail**

1. Unlock clamp by turning handle counterclockwise and remove accessory
2. Slide clamp away from standoffs, then push top of clamp down with thumb until it releases from rail, then grab handle and lift clamp from side rail

### **COMPONENT OVERVIEW**

Stirrup Clamp is a surgical table clamp that attaches to surgical table side rails. Clamp is primarily used in conjunction with our Great White Stirrups.

See side rail dimensions below:

**US:** 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0396

**Europe:** 0.394" x 0.984" (10 mm x 25 mm ) PN# 800-0396-EU

### **GENERAL INFORMATION**

- Product not made with Natural Rubber Latex
- Device supports 1,000 lb. (454 kg) proportional patient load (6'4" (193 cm) tall patient per 99% human body model)
- Product warranty covers product from manufacturing defects for period of 2 years
- If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.
- Product is maintenance-free, check product condition before next use
- Life of device is 5 years under normal use
- Store device between -4°F to +86°F (-20°C to 30°C)

### **DISPOSAL**

- **General** - Prevent infection by cleaning and disinfecting product before disposal
- **Packaging** - Dispose packaging material via household waste according to national requirements
- SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements



### **Product Use Warnings**

#### **WARNING!**

*Maximum load should not exceed appropriate proportion of a patient weighing 1,000 lbs. (454 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.*

#### **Cleaning Recommendation**

*Follow current Association of periOperative Registered Nurses (AORN) Journal Guidelines for proper cleaning and disinfection procedures.*

#### **CAUTION**

*Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.*

- 1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
- 2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
- 3. Let accessory dry*

#### **USER NOTICE**

*Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.*



IFU-800-0396 Rev 3.00