



Instructions for Use

- REF** Schure Socket 800-0006
- REF** Schure Socket (DEN) 800-0006-DEN
- REF** Schure Socket (EU) 800-0006-EU
- REF** Schure Socket (JPN) 800-0006-JPN
- REF** Schure Socket (SWISS) 800-0006-SWISS
- REF** Schure Socket (UK) 800-0006-UK

INTENDED USE

Intended use is to hold operating room accessories with a .625" (1.6 cm) post on a surgical table. The intended users of this device are medical professionals within hospitals and surgery centers.



INSTRUCTIONS

Become familiar with the features of patient positioning device before use with a patient. Always practice use on a nurse, physician or appropriate volunteer prior to using clinically.

Attaching to Side Rail

1. Attach Schure Socket to surgical table side rail with logo sticker side up
2. Insert 5/8" (1.6 cm) round mounting post into mounting hole
3. Secure clamp to lock position by turning handle clockwise

Detaching from Side Rail

1. Unlock clamp by turning handle counterclockwise and remove mounting post

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

- Length: 3" +/- 0.5" (8 cm +/- 1 cm)
- Width: 2" +/- 0.5" (5 cm +/- 1 cm)
- Depth: 8" +/- 0.5" (20 cm +/- 1 cm) (rail handle extended)
- Device Weight: 1.5 +/- 0.5 lbs. (.68 kg +/- .22 kg)
- Attaches to rail of surgical table anywhere on rail
- Single-person installation
- Twist Lock/Release Handle
- Holds accessory equipment with mounting posts up to 5/8" (1.6 cm) in diameter

COMPONENT OVERVIEW

Schure Socket is a surgical table clamp that attaches to all rails. Its function is to hold operating room accessories on a surgical table.

See side rail dimensions below.

US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0006

Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0006-DEN

Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0006-EU

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0006-UK

Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0006-JPN

Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0006-SWISS

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Device supports 500 lb. (227 kg) proportional patient load (6'4" (193 cm) tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



PRODUCT USE WARNINGS

WARNING!

Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. (227 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.



WARNING! *Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.*

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.

CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.
2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory
3. Let accessory dry

USER NOTICE

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

eIFU Language Versions

To download and print the Instructions for Use, please go to www.schuremed.com

Symbol Glossary

Symbol	Title
	Manufacturer
	Date of manufacture
	Authorized Representative in the European Community
	Authorized Representative in the Swiss
	Importer
	Serial Number
	Warning
	Medical Device
	Unique Device Identifier
	CE Marking



SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**

