



## Instructions for Use

**REF** Leg Prepper 800-0013

### ***Replacement Part***

**Leg Prepper Pad and Cover 790-0074**

### **INTENDED USE**

Intended use is to hold patient's leg comfortably during prepping procedure. The intended users of this device are medical professionals within hospitals and surgery centers.

### **INSTRUCTIONS**

Become familiar with patient positioning device's features before use with patient. Always practice use on a nurse, physician or appropriate volunteer prior to using clinically.



### **Attaching Leg Prepper to Table**

1. Snap Schure Socket XL P/N 800-0134 (sold separately) onto the rail
2. Insert mounting post into socket - lock tight

### **Detaching Leg Prepper from Surgical Table**

1. Turn clamp handle counterclockwise to loosen Leg Prepper and remove

### **GENERAL SPECIFICATIONS**

*Device Dimensions (maximum)*

- Length: 36.5" +/- 0.5" (93 cm x 1 cm)
- Width: 12" +/- 0.5" (31 cm +/- 1 cm)
- Depth: 2" +/- 0.5" (5 cm +/- 1 cm)
- Device Weight: 4 +/- 0.5 lbs. (1.8 +/- .22 kg)
- 5/8" (1.56 cm) diameter mounting post inserts into socket
- Single-person installation
- Twist lock/Release Handle

### **COMPONENT OVERVIEW**

Leg Prepper is a surgical table accessory. Padded horseshoe design holds patient's leg comfortably during prepping procedure.

### ***Replacement Part***

**790-0074** Leg Prepper Pad and Cover

**Other required product for use:**

**800-0134 Schure Socket XL** (sold separately)

*US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0134*

*Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0134-DEN*

*Europe: 0.394" x 0.984" (10 mm x 25 mm ) PN# 800-0134-EU*

*Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0134-UK*

*Japan: 0.354" x 1.260" (9 mm x 32 mm ) PN# 800-0134-JPN*

*Swiss: 0.394" x 1.181" (10 mm x 30 mm ) PN# 800-0134-SWISS*

**GENERAL INFORMATION**

- *Product not made with Natural Rubber Latex*
- *Device supports 500 lb. (227 kg) proportional patient load (6'4" (193 cm) tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*

**DISPOSAL**

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*

**PRODUCT USE WARNINGS****WARNING!**

*Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. (227 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.*

**WARNING!**

*Hazards result from incorrect use. Strictly follow Instructions for Use for your Operating Table system.*



**WARNING!** *Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.*

**WARNING!**

*Surgical table load capacities may be less. Never overload a surgical table. Device is intended for mounting on side of surgical table rail only.*

**CLEANING RECOMMENDATION**

*Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.*

**WARNING!**

*Adhere to standards for blood-borne pathogens from Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.*

**CAUTION**

**Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.**

- 1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
- 2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
- 3. Let accessory dry*



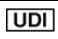


**USER NOTICE**

*Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.*

**eIFU Language Versions**

To download and print the Instructions for Use, please go to [www.schuremed.com](http://www.schuremed.com)

## Symbol Glossary

Symbol	Title
	Manufacturer
	Date of manufacture
	Authorized Representative in the European Community
	Authorized Representative in the Swiss
	Importer
	Serial Number
	Warning
	Medical Device
	Unique Device Identifier
	CE Marking
	Single Patient Use



**SchureMed** (081001460)

452 Randolph Street, Abington, MA 02351 USA

**Toll Free** (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | [orders@schuremed.com](mailto:orders@schuremed.com)

