



## Instructions for Use

**REF** Minor Procedure Table, Phenolic  
800-0026-MI-P


**REF** Minor Procedure Table, Carbon  
Fiber 800-0026-MI-CF

### *Replacement Pad*

Minor Procedure Table Pad 508-1432

### INTENDED USE

Its function is to provide a stable platform for all arm and hand surgeries. The intended users of this device are medical professionals within hospitals and surgery centers.

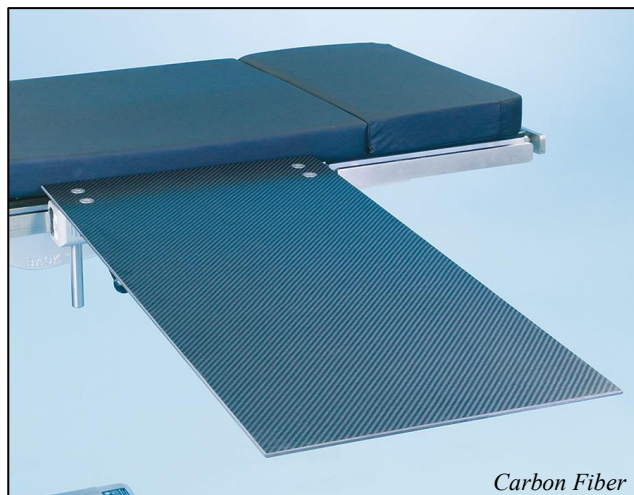

Carbon Fiber

### GENERAL SPECIFICATIONS

#### *Device Dimensions (maximum)*

- *Patient Weight Capacity: 600 lbs. (272 kg), 20 lbs. (9 kg) @ far end of table, 41 lbs. (18.5 kg) in center of table*
- *Length: 30" +/- 0.5" (76 cm +/- 1 cm)*
- *Width: 15" +/- 0.5" (38 cm +/- 1 cm)*
- *Maximum Raised Height: 3.5" +/- 0.5" (9 cm +/- 1 cm)*
- *Phenolic Depth: .375" +/- 0.5" (.95 cm +/- 1 cm)*
- *Carbon Fiber Depth: .25" +/- 0.5" (.64 cm +/- 1 cm)*
- *Phenolic: Device Weight: 10 +/- 0.5 lbs. (4.5 +/- .22 kg)*
- *Carbon Fiber: Device Weight: 9 +/- 0.5 lbs. (4 +/- .22 kg)*
- *Hand Table Mounting Post: 6"L (15 cm)*
- *Attaches to rail of surgical table at any point on rail*
- *Single-person installation*

### GENERAL INFORMATION


- *Product not made with Natural Rubber Latex*
- *Device supports 600 lb. (272 kg) proportional patient load (6'4" (193 cm) tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*

- Store device between -4°F to +86°F (-20°C to 30°C)

## INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician or appropriate volunteer prior to using clinically.

### Attaching Procedure Table to Surgical Table

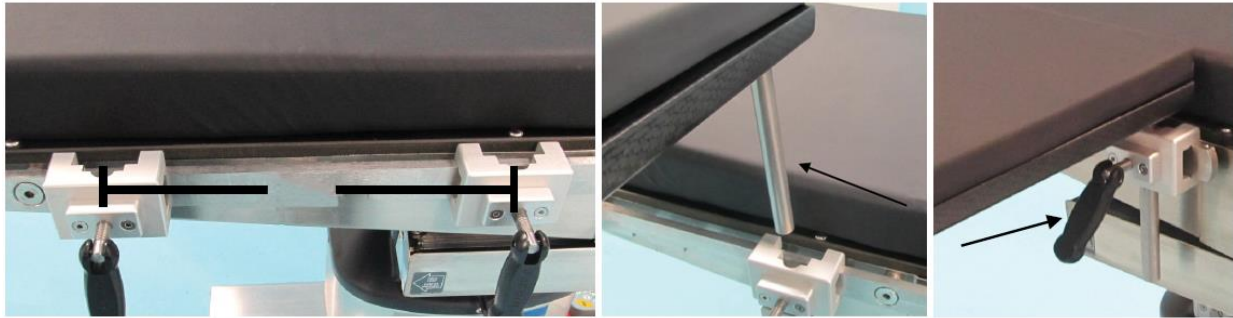

1. Attach clamps (sold separately) to side rail and place them approximately 1' (31 cm) apart (as shown)
2. Insert mounting rods into clamps (as shown)
3. Adjust procedure table so that it is level with surgical table pad. Secure table to accessory rail using the Locking Knobs of the clamps (as shown)
4. Includes 1" (3 cm) pressure management pad
5. Clamps sold separately

## COMPONENT OVERVIEW

Minor Procedure Table, Phenolic or Carbon Fiber are surgical table extensions that attach to all rails.

### Replacement Pad: 508-1432 Minor Procedure Table Pad

**Two Clamps Required: Choose Universal Rail Clamp for US or Deluxe Rail Clamp for Outside US**

#### US – Universal Rail Clamp

- US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0085

#### Outside US – Deluxe Rail Clamp

- Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0248-DEN
- Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0248-EU
- Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0248-UK
- Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0248-JPN
- Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0248-SWISS

## DISPOSAL

- **General** - Prevent infection by cleaning and disinfecting product before disposal
- **Packaging** - Dispose packaging material via household waste according to national requirements
- SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements



## **PRODUCT USE WARNINGS**

### **WARNING!**

*Maximum load should not exceed appropriate proportion of a patient weighing 600 lbs. (272 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.*



**WARNING!** *Hazard resulting from incorrect use. Strictly follow Instructions for Use with your Operating Table system.*



**WARNING!** *Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.*

## **CLEANING RECOMMENDATION**

**Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.**



### **WARNING!**

*Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.*

## **CAUTION**

**Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.**

- 1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
- 2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
- 3. Let accessory dry*




## **USER NOTICE**

*Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.*

## **eIFU Language Versions**

To download and print the Instructions for Use, please go to [www.schuremed.com](http://www.schuremed.com)

## Symbol Glossary

Symbol	Title
	Manufacturer
	Date of manufacture
	Authorized Representative in the European Community
	Authorized Representative in the Swiss
	Importer
	Serial Number
	Warning
	Medical Device
	Unique Device Identifier
	CE Marking



**SchureMed** (081001460)

452 Randolph Street, Abington, MA 02351 USA

**Toll Free** (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | [orders@schuremed.com](mailto:orders@schuremed.com)

