



Instructions for Use

REF Patient Restraint Strap, 96"L 800-0057

REF Bariatric Restraint Straps with Pad 800-0119

Accessory (sold separately)

Restraint Strap Mounting Hook 512-0039

INTENDED USE

Intended use is to stabilize patient positioning for various surgical procedures on operating room table. The intended users of this device are medical professionals within hospitals and surgery centers.

INSTRUCTIONS

Become familiar with features of patient positioning device before use with patient. Always practice use on a nurse, physician or appropriate volunteer prior to using clinically.

Attachment and Set Up

1. Place pad over patient's abdomen area.
2. Place strap(s) over pad and secure to surgical table side rails by looping them around side rails or by using Mounting Hooks (sold separately).
3. If mounting hooks are used, simply loop straps through mounting hook(s).
4. Adjust and tighten by lifting buckles and pulling strap(s) securely.

Removal Instructions

1. Loosen straps by lifting buckles and removing hooks (if used).

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

	800-0057	800-0119
Strap Length	96" +/- 0.5" (243.8 cm +/- 1 cm)	132" +/- 0.5" (353 cm +/- 1 cm)
Strap Width	2" +/- 0.5" (5.1 cm +/- 1 cm)	2" +/- 0.5" (5.1 cm +/- 1 cm)
Pad Length	18" +/- 0.5" (46.7 cm +/- 1 cm)	30" +/- 0.5" (76.2 cm +/- 1 cm)
Pad Width	5" +/- 0.5" (12.7 cm +/- 1 cm)	12" +/- 0.5" (30.5 cm +/- 1 cm)
Device Weight	1.8 +/- 0.5 lbs. (1 +/- .22 kg)	5.5 +/- 0.5 lbs. (2.5 +/- .22 kg)



COMPONENT OVERVIEW

Restraint Straps are a surgical table accessory that stabilizes patient positioning for various surgical procedures on operating room table.

Accessory (sold separately)

Restraint Strap Mounting Hook 512-0039

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*
- *Clean with hospital grade disinfectant*

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



PRODUCT USE WARNINGS

WARNING!

Hazard resulting from incorrect use. Strictly follow Instructions for Use with your surgical table system.



WARNING!

Take care as pinching of the skin may occur when opening and closing strap buckles.



WARNING!

Do not use equipment if worn, damaged, or if pieces are missing.



WARNING! *Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.*

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.

**WARNING!**

Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.

CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

- 1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
- 2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
- 3. Let accessory dry*

USER NOTICE

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

eIFU Language Versions

To download and print the Instructions for Use, please go to www.schuremed.com

Symbol Glossary

Symbol	Title
	Manufacturer
	Date of manufacture
	Authorized Representative in the European Community
	Authorized Representative in the Swiss
	Importer
	Serial Number
	Warning
	Medical Device
	Unique Device Identifier
	CE Marking



SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**

