



## Instructions for Use

**REF** Disposable Face Masks 800-0164

### INTENDED USE

Intended use is to hold patient's head during operative procedure. The intended users of this device are medical professionals within hospitals and surgery centers.

### INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician or appropriate volunteer prior to using clinically.



1. Place patient on surgical table/shoulder chair in supine position
2. After intubating patient, hold patient's head/neck and raise shoulder chair up so patient is in Fowler (sitting) position
3. Loosen ball joint and adjust head positioner around patient's head. Make sure patient's ears are uncovered. Then tighten ball joint.
4. Holding head positioner flaps alongside patient's head, place face mask over face as shown - making sure eyes, nose and mouth are uncovered
5. Affix top and bottom VELCRO® straps of mask on one side to head positioner. Then affix other side as shown - making sure facemask is centered over face.



**NOTE:** Head should be securely positioned. If not, make straps a little tighter.

### GENERAL SPECIFICATIONS

*Device Dimensions (maximum)*

- Length: **Strap:** 29" +/- 0.5" (74 cm +/- 1 cm), **Face Mask:** 9" +/- 0.5" (23 cm +/- 1 cm)
- Width: 13.5" +/- 0.5" (34 cm +/- 1 cm)
- Depth: 1" +/- 0.5" (2.5 cm +/- 1 cm)
- Device Weight: 0.25 +/- 0.5 lbs. (.11 +/- .22 kg)
- Attaches to head section of SchureMed Shoulder Chairs
- Single-person installation

### COMPONENT OVERVIEW

Disposable Face Mask securely holds a patient's head during operative procedure. Straps are soft and pliable to attach to head frame.

## GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Store device between -4°F to +86°F (-20°C to 30°C)*



**SINGLE PATIENT USE—Please dispose after each use**

## USER NOTICE

*Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.*

## eIFU Language Versions

To download and print the Instructions for Use, please go to [www.schuremed.com](http://www.schuremed.com)

## Symbol Glossary

Symbol	Title
	Manufacturer
	Date of manufacture
	Authorized Representative in the European Community
	Authorized Representative in the Swiss
	Importer
	Catalogue Number
	Serial Number
	Warning
	Medical Device
	Unique Device Identifier
	CE Marking
	Single Patient Use



**SchureMed** (081001460)

452 Randolph Street, Abington, MA 02351 USA

**Toll Free** (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | [orders@schuremed.com](mailto:orders@schuremed.com)

