



Instructions for Use

- REF** Deluxe Rail Clamp 800-0248
- REF** Deluxe Rail Clamp (EU) 800-0248-EU
- REF** Deluxe Rail Clamp (UK) 800-0248-UK
- REF** Deluxe Rail Clamp (DEN) 800-0248-DEN
- REF** Deluxe Rail Clamp (JPN) 800-0248-JPN
- REF** Deluxe Rail Clamp (SWISS) 800-0248-SWISS

INTENDED USE

Intended use is to hold surgical accessories on a surgical table. The intended users of this device are medical professionals within hospitals and surgery centers.

INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician or appropriate volunteer prior to using clinically.



Deluxe Rail Clamp attaches anywhere along surgical table side rail. It accepts both 5/8" (1.6 cm) diameter round mounting posts and 1" x 3/8" (2.5 cm x .95 cm) flat bar mounting posts. Holds accessories in vertical or horizontal position.

Attaching to Side Rail

1. Attach clamp on rail
2. Place mounting post into mounting hole
3. Turn clamp handle clockwise to securely lock

Detaching from Side Rail

1. Turn clamp handle counter-clockwise to unlock and remove from rail

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

- Length: 3" +/- 0.5" (8 cm +/- 1 cm) (Handle Collapsed)
- Width: 2" +/- 0.5" (5 cm +/- 1 cm) (Handle Extended)
- Depth: 4" +/- 0.5" (10 cm +/- 1 cm) (with rail handle)
- Device Weight: 1 +/- 0.5 lbs. (.45 kg +/- .22 kg)
- Attaches to rail of surgical table at any point on rail
- Single-person installation

- *Holds accessory equipment up to 1 x 3/8" (2.5 cm x .95 cm) flat mounting post and round mounting posts up to 5/8" (1.6 cm)*

COMPONENT OVERVIEW

Deluxe Rail Clamp is a surgical table clamp that attaches to surgical table side rails. Its function is to hold surgical accessories on a surgical table.

See side rail dimensions below:

US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0248

Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0248-DEN

Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0248-EU

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0248-UK

Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0248-JPN

Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0248-SWISS

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Device supports 500 lb. (227 kg) proportional patient load (6'4" (193 cm) tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



PRODUCT USE WARNINGS

WARNING! *Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. (227 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.*



WARNING!

Hazards result from incorrect use. Strictly follow Instructions for Use with your Operating Table system.



WARNING! *Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.*

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.



WARNING!

Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.

CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.
2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory
3. Let accessory dry

USER NOTICE

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

eIFU Language Versions

To download and print the Instructions for Use, please go to www.schuremed.com

Symbol Glossary

Symbol	Title
	Manufacturer
	Date of manufacture
	Authorized Representative in the European Community
	Authorized Representative in the Swiss
	Importer
	Batch Code
	Serial Number
	Warning
	Medical Device
	Unique Device Identifier
	CE Marking



SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**

