



## Instructions for Use

**REF** Pediatric E-Z Lift Stirrups, KIDS  
800-0342-KIDS

**REF** Pediatric E-Z Lift Stirrups, PUPS  
800-0342-PUPS

### *Replacement Pads*

**KIDS Boot Pads 508-1296**

**PUPS Boot Pads 508-1295**

### **INTENDED USE**

Pediatric E-Z Lift Stirrups are a patient positioning accessory used in Gynecology, Urology, and laparoscopic procedures. They provide a safe system for positioning the legs up to 160 lb. (73 kg) and 250 lb. (113 kg) patients in a variety of lithotomy/abduction positions. The intended users of this device are medical professionals within hospitals and surgery centers.



### **GENERAL SPECIFICATIONS**

#### *Device Dimensions (maximum)*






- *Height: 35" +/- 0.5" (89 cm +/- 1 cm)*
- *Width: 9" +/- 0.5" (23 cm +/- 1 cm)*
- *Depth: 14" +/- 0.5" (36 cm +/- 1 cm)*
- *Device Weight Per Stirrup: 8 +/- 0.5 lbs. (3.6 +/- .22 kg) (with pad)*
- *Range of Motion: -30° to 75° lithotomy range, -9° to 25° adduction to abduction*
- *Single-person installation*
- *Connects to seat section of surgical table*
- *Store stirrups on transport/storage dolly (P/N 800-0074-P sold separately)*

### **GENERAL INFORMATION**

- *Product not made with Natural Rubber Latex*
- *This device supports a 160 lb. (73 kg) patient for 800-0342-PUPS and 250 lb. (113 kg) patient for 800-0342-KIDS proportional patient load (5'4" (163 cm) tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*

## INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician or appropriate volunteer prior to using clinically.

<p>1. Attach SpringLoc Clamps P/N 800-0338 on accessory rails in same locations on opposite sides of surgical table at patient's hip joints.</p>	
<p>2. Prior to placing device into SpringLoc Clamp, identify patient's left and right side stirrup indicated on boot label.</p>	
<p>3. Insert stirrup blade into clamp. Tighten clamp by turning handle to the right.</p>	
<p>4. Ensure patient is positioned on surgical table in accordance with procedure and surgeon requirements.  <b>WARNING!! To prevent patient or operator injury from inadvertent stirrup movement, securely tighten accessory clamp and boot clamp.</b></p>	
<div style="display: flex; justify-content: space-around; align-items: flex-end;">    </div> <p style="text-align: center;">(1) Trigger Handle (2) Free Floating Boot Pivot (3) Boot Clamp</p>	
<p>5. Support patient's leg by grasping heel in one hand and underside of knee with other hand. Gently flex knees and transfer leg into boot, then secure boot straps.</p> <p>6. To achieve appropriate leg/foot position, loosen boot clamp. Adjust boot to desired position and re-tighten clamp. Ensure patient's heels are securely seated in boot heels.</p> <p>7. To achieve appropriate lithotomy and abduction positions, squeeze trigger, adjust to desired position and release to lock.</p>	

**Note:** Free-floating boot is designed to rotate about pivot shown above through-out the full range of motion, reducing risk of superficial nerve injury.

## TAKE DOWN

1. Loosen clamps and remove stirrups by lifting them out of clamps
2. Remove accessory clamps from side rails

## COMPONENT OVERVIEW

Pediatric E-Z Lift Stirrups provide a safe system for positioning legs of up to 160 lbs. (73 kg) per patients for 800-0342-PUPS and 250 lbs. (113 kg) per patient for 800-0342-KIDS in a variety of lithotomy/abduction positions.

### *Other required products for use:*

**800-0338** SpringLoc Clamp

**800-0338-EU** SpringLoc Clamp

**800-0338-JPN** SpringLoc Clamp

**800-0338-SWISS** SpringLoc Clamp

### *Replacement Pads*

**508-1296** KIDS Boot Pads

**508-1295** PUPS Boot Pads

## DISPOSAL

- **General** - Prevent infection by cleaning and disinfecting product before disposal
- **Packaging** - Dispose packaging material via household waste according to national requirements
- SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements



### PRODUCT USE WARNINGS

**WARNING!** Maximum load should not exceed appropriate proportion of a patient weighing 160 lbs. (73 kg) for 800-0342-PUPS and 250 lbs. (113 kg) for 800-0342-KIDS. Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.



**WARNING!** Hazards result from incorrect use. Strictly follow instructions for use with your Operating Table system.



**WARNING!** Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.



**WARNING!** Surgical table load capacities may be less. Never overload a surgical table. Device is intended for mounting on surgical table side rail only.

## CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedure.



### **WARNING!**

*Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.*

## CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.
2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory
3. Let accessory dry

## USER NOTICE

*Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.*

## eIFU Language Versions

To download and print the Instructions for Use, please go to [www.schuremed.com](http://www.schuremed.com)

## Symbol Glossary

Symbol	Title
	Manufacturer
	Date of manufacture
	Authorized Representative in the European Community
	Authorized Representative in the Swiss
	Importer
	Serial Number
	Warning
	Medical Device
	Unique Device Identifier
	CE Marking



**SchureMed** (081001460)

452 Randolph Street, Abington, MA 02351 USA

**Toll Free** (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**

