



Instructions for Use

REF	Infinity Clamp	800-0386
REF	InfinitySQ	800-0386-SQ
REF	Infinity Blade Clamp	800-0386-F

800-0386



800-0386-SQ



INTENDED USE

Intended use is to hold surgical accessories on surgical table. The intended users of this device are medical professionals within hospitals and surgery centers.

INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice use on a nurse, physician or appropriate volunteer prior to using clinically.

Fits all standard operating room table side rails. Infinity Clamp accepts round mounting posts from 5/8" to 3/4" (1.6 cm) in diameter and rectangular mounting posts 1" x up to 3/8" (2.5 cm x .95 cm). Infinity Clamp 20MM Square accepts .787" (20mm) square rod. Infinity Clamp Fixed accepts round mounting posts from 5/8" to 3/4" (1.9 cm) in diameter and rectangular mounting posts 1" x up to 3/8" (2.5 cm x .95 cm).

800-0386-F



Attaching Clamp to Surgical Table

1. Attach Infinity Clamp to surgical table side rail with logo sticker side up
2. Place round or flat post in mounting hole
3. To tighten post, turn handle clockwise until locked

Detaching Clamp from Surgical Table

1. Turn handle counterclockwise until post loosens
2. Remove post from socket
3. Detach from rail

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

	<u>800-0386</u>	<u>800-0386-SQ</u>	<u>800-0386-F</u>
Length	4.75" +/- 0.5" (12 cm +/- 1 cm)	4.75" +/- 0.5" (12 cm +/- 1 cm)	4.0" +/- 0.5" (10 cm +/- 1 cm)

Width	2.65" +/- 0.5" (6.7 cm +/- 1 cm)	2.65" +/- 0.5" (6.7 cm +/- 1 cm)	2.65" +/- 0.5" (6.7 cm +/- 1 cm)
Height	3.88" +/- 0.5" (10 cm +/- 1 cm)	3.88" +/- 0.5" (10 cm +/- 1 cm)	3.88" +/- 0.5" (10 cm +/- 1 cm)
Device Weight	1.5 +/- 0.5 lbs. (.68 +/- .22 kg)	1.5 +/- 0.5 lbs. (.68 +/- .22 kg)	1.3 +/- 0.5 lbs. (.58 +/- .22 kg)

- *Attaches to surgical table rail at any point on rail*
- *Single-person installation*
- *Twist lock/release handle*

COMPONENT OVERVIEW

Infinity Clamp is a surgical table clamp that attaches to side rails of any table.

See side rail dimensions below:

US: 0.374" x 1.122" (9.5 mm x 28.5 mm)

Denyer: 0.236" x 1.496" (6 mm x 38 mm)

Europe: 0.394" x 0.984" (10 mm x 25 mm)

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm)

Japan: 0.354" x 1.260" (9 mm x 32 mm)

Swiss: 0.394" x 1.181" (10 mm x 30 mm)

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Device supports 800 lb. (363 kg) proportional patient load (6'4" (193 cm) tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



PRODUCT USE WARNINGS

WARNING! *Maximum load should not exceed appropriate proportion of a patient weighing 800 lbs. (363 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded. Strictly follow Instructions for Use for your surgical table system.*



WARNING! Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.

CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.
2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory
3. Let accessory dry

USER NOTICE

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

eIFU Language Versions

To download and print the Instructions for Use, please go to <https://www.schuremed.com>, select the product and click on the drop-down menu for currently available translations for the IFU.

Symbol Glossary

Symbol	Title
	Manufacturer
	Authorized Representative in the European Community
	Use-by Date
	Batch Code
	Catalogue or Item Number
	Serial Number
	Caution
	Medical Device
	Unique Device Identifier
	CE Marking



SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | orders@schuremed.com

