



Instructions for Use



Standard Armboard

- REF** Standard Armboard 800-0008
- REF** Standard Armboard (DEN)
800-0008-DEN
- REF** Standard Armboard (EU)
800-0008-EU
- REF** Standard Armboard (JPN)
800-0008-JPN
- REF** Standard Armboard (SWISS)
800-0008-SWISS
- REF** Standard Armboard (UK)
800-0008-UK

Pad sold separately: Choose one
2" Armboard Pad, Deluxe 508-0071
2" Armboard Pad, Softcare 508-0151



Extra-Wide Armboard

- REF** Extra-Wide Armboard 800-0018
- REF** Extra-Wide Armboard (DEN)
800-0018-DEN
- REF** Extra-Wide Armboard (EU)
800-0018-EU
- REF** Extra-Wide Armboard (JPN)
800-0018-JPN
- REF** Extra-Wide Armboard (SWISS)
800-0018-SWISS
- REF** Extra-Wide Armboard (UK)
800-0018-UK

Pad sold separately: Choose One
2" Extra-Wide Armboard Pad, Deluxe
508-0105
2" Extra-Wide Armboard Pad, Softcare
508-0152

INTENDED USE

Intended use is to hold patient's arm - range of motion is 180°. The intended users of this device are medical professionals within hospitals and surgery centers.

INSTRUCTIONS FOR USE

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician or appropriate volunteer prior to using clinically.

Attaching Armboard to Table

- *Attaches to any point on the surgical table side rail*

Adjusting Armboard

- *Pull Directional Trigger (outer most trigger), rotate to desired position, release to lock*

Detaching Armboard from Surgical Table

- *Press Release Trigger (trigger closest to table connection) and lift armboard from table*

GENERAL SPECIFICATIONS

- *Attaches to rail of surgical table at any point on rail*
- *Single-person installation*
- *Push Release Trigger*

	Standard Armboard 800-0008	Extra-Wide Armboard 800-0018
<i>Length</i>	<i>26.5" +/- 0.5" (67 cm +/- 1 cm)</i>	<i>26.5" +/- 0.5" (67 cm +/- 1 cm)</i>
<i>Width</i>	<i>6" +/- 0.5" (15 cm +/- 1 cm)</i>	<i>11" +/- 0.5" (28 cm +/- 1 cm)</i>
<i>Depth</i>	<i>2.5" +/- 0.5" (6 cm +/- 1 cm) with pad</i>	<i>2.5" +/- 0.5" (6 cm +/- 1 cm) with pad</i>
<i>Device Weight</i>	<i>5.25 +/- 0.5 lbs. (2.4 kg +/- .22 kg)</i>	<i>9 +/- 0.5 lbs. (4 +/- 0.22 kg)</i>

COMPONENT OVERVIEW

Standard and Extra-Wide Armboards are surgical table extensions that attach to side rails of any table.

See side rail dimensions below:

US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0008 & 800-0018

Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0008-DEN & 800-0018-DEN

Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0008-EU & 800-0018-EU

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0008-UK & 800-0018-UK

Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0008-JPN & 800-0018-JPN

Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0008-SWISS & 800-0018-SWISS

Standard Armboard 800-0008

Pad sold separately: Choose one

2" Armboard Pad, Deluxe 508-0071

2" Armboard Pad, Softcare 508-0151

Extra-Wide Armboard 800-0018

Pad sold separately: Choose one

2" Extra-Wide Armboard Pad, Deluxe 508-0105

2" Extra-Wide Armboard Pad, Softcare 508-0152

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Device supports 500 lb. (227 kg) proportional patient load (6'4" (193 cm) tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*

- *If damaged in shipping, call Customer Service at (888) 724-8763 or (781) 982-7000 to obtain Return Material Authorization number. For product warranty issues, contact Customer Service.*
- *CE marked medical device according to MDR (EU) 2017/745*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F (-20°C to 30°C)*

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



PRODUCT USE WARNINGS

WARNING!

Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. (227 kg). Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.



WARNING!

Hazards result from incorrect use. Strictly follow Instructions for Use with your Operating Table system.



WARNING! *Do not reuse device if there are obvious signs of damage or functional issues. Consult manufacturer before reusing.*

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.



WARNING!

Always follow the standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use the recommended protective clothing, gloves, masks and eye protection to clean the accessory.

CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

1. *Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
2. *Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
3. *Let accessory dry*

USER NOTICE

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

eIFU Language Versions

To download and print the Instructions for Use, please go to www.schuremed.com

Symbol Glossary

Symbol	Title
	Manufacturer
	Date of manufacture
	Authorized Representative in the European Community
	Authorized Representative in the Swiss
	Importer
	Serial Number
	Warning
	Medical Device
	Unique Device Identifier
	CE Marking



SchureMed (081001460)

452 Randolph Street, Abington, MA 02351 USA

Toll Free (888) 724-8763 | **Ph** (781) 982-7000 | **Fax** (781) 982-7001 | **orders@schuremed.com**

