



Schure Socket

#800-0006
#800-0006-DEN

#800-0006-EU
#800-0006-JPN

#800-0006-UK
#800-0006-SWISS

Operating room staff can quickly position surgical devices anywhere along standard operating room table side rails - Use the Schure Socket to prevent all movement on mounting posts

NEXT
Generation



Accepts round mounting posts up to 5/8" (1.6 cm) in diameter

Patient weight capacity is 500 lbs (227 kg)

The industrial-strength Schure Socket fits all operating room tables with standard 3/8"W x 1-1/8"H (.95 cm x 2.85 cm) side rails

Connects anywhere along side rail, without the need for notches



To connect, place top lip over side rail with logo facing up



Tilt clamp down until bottom lip is underneath side rail and you hear a click



To disconnect, push in on the clamp to unlock



Tilt clamp up and pull away from the side rail

For use with:

